
RSWT® - RADIAL SHOCK WAVE THERAPY  
THÉRAPIE PAR ONDES DE CHOC RADIALES  
RADIALE STOSSWELLENTHERAPIE  
TERAPIA POR ONDAS DE CHOQUE RADIALES  
TERAPIA AD ONDE D'URTO RADIALI  
RADIELL STÖTVÅGSTERAPI

Swiss DolorClast®  
product range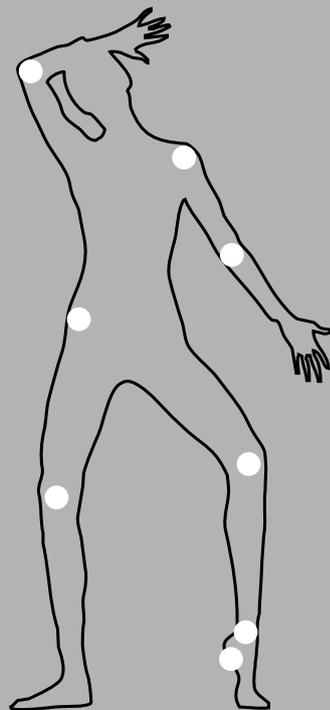

**Rx ONLY** Caution! Federal (USA) law restricts this device  
to sale by or on the order of a physician.

**EMS**   
ELECTRO MEDICAL SYSTEMS

**Swiss DolorClast®**  
product range

**RSWT - RADIAL SHOCK WAVE THERAPY®**  
**THÉRAPIE PAR ONDES DE CHOC RADIALES**  
**RADIALE STOSSWELLENTHERAPIE**  
**TERAPIA POR ONDAS DE CHOQUE RADIALES**  
**TERAPIA AD ONDE D'URTO RADIALI**  
**RADIELL STÖTVÅGSTERAPI**

## DEAR CUSTOMERS,

Thank you for buying a new EMS product. It meets the highest quality and safety standards.

The following indications have been successfully treated with the EMS Swiss DolorClast method since 1998:

- Calcifying tendinitis of the shoulder
- Subacromial pain syndrome
- Tennis elbow (epicondylitis humeri radialis)
- Greater trochanteric pain syndrome
- Patella tip syndrome
- Rugby knee (Osgood-Schlatter disease)
- Medial tibial stress syndrome
- Insertional Achilles tendinopathy
- Midbody Achilles tendinopathy
- Plantar fasciitis (or fasciopathy)
- Myofascial pain syndrome (trigger point therapy), idiopathic low back pain, and pseudoradicular syndromes
- Peyronie’s disease (induratio penis plastica)
- Acute and chronic soft-tissue wounds (including diabetic foot ulcers)
- Primary and secondary lymphedema
- Cellulite

Several scientific publications and empirical reports have proven clinical efficacy and safety of the EMS Swiss DolorClast method for these indications.

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**Contraindications:**

- Treatment over air-filled tissue (lung, gut)
- Treatment of pre-ruptured tendons
- Treatment of pregnant women
- Treatment of patients under the age of 18 (except for the treatment of Osgood-Schlatter disease)
- Treatment of patients with blood-clotting disorders (including local thrombosis)
- Treatment of patients treated with oral anticoagulants
- Treatment of tissue with local tumors or local bacterial and/or viral infections
- Treatment of patients treated with cortisone

**Risks:**

- Pain and discomfort during and after treatment (anesthesia is not necessary)
- Reddening of the skin
- Petechia
- Swelling and numbness of the skin over the treatment area
- Potential damage of lung and gut tissue
- Potential rupture of pre-ruptured tendons
- Potential damage of immature epiphyses
- Possible dissemination of malignant tumors
- Potential damage of articular cartilage

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## ABOUT THIS MANUAL

These operating instructions describe the treatment of chronic soft tissue pain near bones, myofascial pain syndromes, wound healing, lymphedema, and treatment of cellulite with the EMS Swiss DolorClast method.

Please read these operating instructions carefully as they explain all the most important details and procedures. Please pay special attention to the safety precautions.

 Instructions concerning the components of the system are provided separately in the related instructions for use. Read the instruction manuals delivered with each of the components before using this product. Do not hesitate to contact your local EMS dealer for further information.

Always keep this instruction close to hand.

To prevent injury to people and damage to property, please heed the relevant warnings and remarks. They are marked as follows:

**Danger**  
Risk of injury

**Caution**  
Risk of property damage or environmental harm

**Note**  
Useful additional information and hints

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**Förbjudet**

**Tillåtet**

Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please call directly in your approved EMS repair center or your dealer.

We wish you lots of success!

EMS

#### Technical modifications

EMS reserves the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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## CONTENT

RSWT - Radial Shock Wave Therapy .....	2
Application on the musculoskeletal system .....	2
Calcifying tendinitis of the shoulder .....	4
Subacromial pain syndrome .....	6
Tennis elbow (Epicondylitis humeri radialis) .....	8
Greater trochanteric pain syndrome .....	10
Patella tip syndrome .....	12
Rugby knee (Osgood-Schlatter disease) .....	14
Medial tibial stress syndrome .....	16
Insertional Achilles tendinopathy .....	18
Midbody Achilles tendinopathy .....	20
Plantar fasciitis / fasciopathy (heel pain) .....	22
Myofascial pain syndrome - Trigger point therapy .....	24
Trigger point therapy: Myofascial pain syndrome.....	26
Idiopathic low back pain .....	28
Pseudoradicular pain syndrome .....	30
Other applications .....	32
Peyronie's disease .....	34
Acute and chronic soft tissue wounds (diabetic foot ulcer as an example) .....	36
Primary and secondary lymphedema.....	40
Cellulite.....	42
Safety precautions .....	44

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## RSWT - RADIAL SHOCK WAVE THERAPY

Radial Shock Wave Therapy (RSWT) has revolutionized pain management in orthopaedics, and is an emerging technology in aesthetic dermatology. The procedure is non-invasive and performed on an outpatient basis.

In pain management, RSWT is usually applied in three treatment sessions, each one week apart. Approximately 80% of all patients report a substantial improvement of the clinical situation already after the second treatment session. According to the most recent studies, this is a positive sign for the prognosis of the treatment outcome. Therefore, it is possible to foresee the treatment outcome after two treatment sessions, but not before.

 The application settings outlined in this manual (handpiece, applicator, number of sessions, number of impulses per treatment, frequency, air pressure) are non-binding recommendations. They are based on reports by RSWT experts published in the scientific literature and presented at international congresses.

 These recommendations should be considered as well as the individual experience of the therapist and the individual feedback of the patient when planning and performing treatments with the Swiss DolorClast method.

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 When performing treatments on the musculoskeletal system as described here, the air pressure of the Swiss DolorClast should be adjusted so that the patient experiences some level of discomfort at least during the first treatment, but does not feel pain. The discomfort indicates that the peripheral nervous system is stimulated by the shock waves. During the following treatments the patient might not experience discomfort anymore, even if the air pressure is increased.

 Please make sure that all side effects from the previous treatment have subsided before performing the next treatment. For patients with a low pain threshold, the treatment can be started at an operating pressure of 2 bar and operating frequency of 3 Hz. An analgesic effect sets in after approximately 500 pulses. As treatment goes on, the working pressure and operating frequency can then be increased to the recommended values. If the sensitivity of the patient to pain does not allow this, continue with the initial settings.

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## APPLICATION ON THE MUSCULOSKELETAL SYSTEM

- For treatments of the musculoskeletal system, localize the region to be treated by palpation (i.e., by localizing the site of maximum pain).
- When treating calcifying tendinitis of the shoulder, the zone to be treated is also identified by a radiograph and/or an echography of the calcified area.
- After localization, label the skin area over the region to be treated.
- Activate the pulses in continuous mode and move the applicator tip over the treated zone. The manual application force depends on the indication.

 Apply the EMS Swiss DolorClast coupling gel for improved coupling.

 Use the appropriate applicator according to the user manual for the Swiss DolorClast applicators.

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## Calcifying tendinitis of the shoulder

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## Kalkaxel

Literature: Magosch et al., Z Orthop 2003;141:629-633 (in PubMed)

- 1 Palpation of the painful region
- 2 Labeling of the painful region
- 3 Application of coupling gel
- 4 Treatment with the Evo Blue or Power+ handpiece
  - Applicator: 15 mm
  - Number of sessions: 3 (at weekly intervals)
  - Impulses per treatment: 2'000
  - Frequency: 5 - 20 Hz
  - Air pressure: Evo Blue: 2 - 4 bar / Power+ : 1.5 - 3 bar

 Exert the maximum application force tolerated by the patient.

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Literature: Mehra et al. Surgeon 2003;1:290-292 (in PubMed)

- 1 Palpation of the painful region
- 2 Labeling of the painful region
- 3 Application of coupling gel
- 4 Treatment with the Evo Blue handpiece
  - Applicator: 15 mm
  - Number of sessions: 3 (at weekly intervals)
  - Impulses per treatment: 2'000
  - Frequency: 5 - 20 Hz
  - Air pressure: 1.5 - 3 bar

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## Greater trochanteric pain syndrome

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- 1 Palpation of the painful region
- 2 Labeling of the painful region
- 3 Application of coupling gel
- 4 Treatment with the Evo Blue handpiece
  - Applicator: 15 mm or 36 mm Planar
  - Number of sessions: 3 (at weekly intervals)
  - Impulses per treatment: 2'000
  - Frequency: 5 - 20 Hz
  - Air pressure: 3 - 4 bar (with both applicators)

 Exert a moderate application force.

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Literature: Lohrer et al., Sportverl Sportschad 2002;16:108-114 ; Furia et al. Knee Surg Sports Traumatol Arthrosc 2012 May 25 (Epub ahead of print) (both in PubMed)

- 1 Palpation of the painful region
- 2 Labeling of the painful region
- 3 Application of coupling gel
- 4 Treatment with the Evo Blue handpiece
  - Applicator: 15 mm
  - Number of sessions: 3 (at weekly intervals)
  - Impulses per treatment: 2'000
  - Frequency: 5 - 15 Hz
  - Air pressure: 2 - 3 bar

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## Rugby knee (Osgood-Schlatter disease)

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## Osgood-Schlatters sjukdom

Literature: Titov and Litvinenko, Abstracts 10th International Congress of the International Society for Musculoskeletal Shock Wave Therapy (ISMST), Toronto, Canada, 2007, pp. 46-47

- 1 Palpation of the painful region
- 2 Labeling of the painful region
- 3 Application of coupling gel
- 4 Treatment with the Evo Blue handpiece
  - Applicator: 15 mm
  - Number of sessions: 3 (at weekly intervals)
  - Impulses per treatment: 2'000-3'000
  - Frequency: 5 - 15 Hz
  - Air pressure: 2 - 3 bar

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Literature: Rompe et al., Am J Sports Med 2010;38:125-132 (in PubMed)

- 1 Palpation of the painful region
- 2 Labeling of the painful region
- 3 Application of coupling gel
- 4 Treatment with the Evo Blue handpiece
  - Applicator: 15 mm
  - Number of sessions: 3 (at weekly intervals)
  - Impulses per treatment: 2'000
  - Frequency: 5 - 20 Hz
  - Air pressure: 2.5 - 3 bar

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## Insertional Achilles tendinopathy

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## Akillestendinit i senfästet

Literature: Rompe et al., J Bone Joint Surg Am 2008;90:52-61 (in PubMed)

- 1 Palpation of the painful region
- 2 Labeling of the painful region
- 3 Application of coupling gel
- 4 Treatment with the Evo Blue handpiece
  - Applicator: 15 mm
  - Number of sessions: 3 (at weekly intervals)
  - Impulses per treatment: 2'000
  - Frequency: 5 - 20 Hz
  - Air pressure: 2 - 3 bar

 Exert a moderate application force from dorsal, medial and lateral directions.

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## Myofascial pain syndrome - Trigger point therapy

 Use of the EMS Swiss DolorClast method in the treatment of myofascial pain syndromes requires specialist training with proof of certification. Please contact the manufacturer for further details on training events.

 Without in-depth training in diagnosis and treatment, the use of the EMS Swiss DolorClast method may be dangerous to the patient and will afford no guarantee of the treatment results being successful.

 A correct diagnosis must be established before applying the shock waves!

The symptoms and trigger point locations may not be in the same part of the body. Symptom-oriented treatment is not successful with trigger point related pain syndromes.

The recommended operating parameters for the various trigger points in the body and the precautions for preventing complications are explained at the certification seminars.

The pulse count to be applied to each trigger point depends on the size and depth of a trigger point.

The operating parameters vary according to the area of use and the muscle region. Please note that no more than 4'000 – 6'000 pulses should be applied per session.

The treatment area is determined before hand by means of a trigger point diagnosis. An ultrasound diagnosis can also be carried out to differentiate between the muscle layers and to establish the precise location within the muscle. Please use biofeedback of the patient to identify the affected muscle.

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## OTHER APPLICATIONS

- Where applicable, localize the region to be treated by palpation (i.e., by localizing the site of maximum pain).
- After localization, mark the skin area over the region to be treated.
- Use the EMS Swiss DolorClast coupling gel for improved coupling.
-  Use a sterile coupling gel when treating soft tissue wounds.
- Activate the pulses in continuous mode and move the applicator tip over the treated zone. The manual application force depends on the indication.

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**Peyronies sjukdom  
(Induratio penis plastica)**

Literature: Haupt et al., Abstracts 19th World Congress on Endourology and SWL, Bangkok, Thailand, 2001

- 1 Palpation of the painful region
- 2 Labeling of the painful region
- 3 Application of coupling gel
- 4 Treatment with the Evo Blue handpiece
  - Applicator: 15 mm
  - Number of sessions: 3 (at weekly intervals)
  - Impulses per treatment: 2'000
  - Frequency: 10 - 20 Hz
  - Air pressure: 2 - 3 bar

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## Acute and chronic soft tissue wounds (diabetic foot ulcer as an example)

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- 1 Perform a standard wound cleaning and debridement
- 2 Place a sterile plastic film on the wound and the surrounding tissue (the film should exceed the wound edges by at least 5 cm)
- 3 Apply sterile coupling gel to the film
- 4 Treatment with the Evo Blue or Power+ handpiece
  - Applicator: 36 mm Planar
  - Number of sessions: 6-8 (twice a week)
  - Impulses per treatment: 1'000/cm<sup>2</sup> of wound
  - Frequency: 5 - 20 Hz
  - Air pressure: 2 - 4 bar

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**After the treatment:**

- 1 Remove the plastic film and the coupling gel
- 2 Clean with sterile saline solution
- 3 Apply standard wound dressing according to the individual phase of wound healing

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Literature: Michelini et al., Eur J Lymphol 2008;19:10

 No need for palpation and labeling because the treatment area can be identified by visual inspection.

1 Application of coupling gel

2 Treatment with the Evo Blue or Power+ handpiece

- Applicator: 36 mm Planar
- Number of sessions: 10 (twice a week)
- Impulses per treatment: 200 - 300 per cm<sup>2</sup> of fibrotic skin
- Frequency: 4 - 20 Hz
- Air pressure: 2 - 4 bar

 Exert a light to moderate application force.

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Literature: EMS (ongoing clinical study)

👉 No need for palpation and labeling because the treatment area can be identified by visual inspection.

- 1 Application of coupling gel
- 2 Treatment with the Evo Blue or Power+ handpiece
  - Applicator: 36 mm Planar
  - Number of sessions: 8 (twice a week)
  - Impulses per treatment: 200 -300 per cm<sup>2</sup> of cellulite skin (~3'500 per skin region)
  - Frequency: 8 - 20 Hz
  - Air pressure: 3 - 4 bar

👉 Exert a light to moderate application force.

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## SAFETY PRECAUTIONS

EMS and the distributor of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operating instructions, or improper preparation and maintenance.

 For further details on the preparing, operating and maintenance of this product, please refer to the relevant operating instructions of the unit and handpiece kit.

 Use for the intended purpose only: Before using the product, make sure that you have studied and understood the operating instructions. This also applies to any equipment used with this product. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

 Risk of explosion: Do not use this product in the presence of inflammable anaesthetics or gases.

 This product must be used only by trained and qualified personnel.

 Before each use, always clean, disinfect, rinse and sterilize the different pieces and accessories of this product. Please refer to the information provided in these operating instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

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MTR - Health & Spa AG