4-Series

Information Booklet
Informations-Booklet
Brochure d'informations
Opuscolo informativo
Folletto de información
Informatieboekje

Copyright:

Enraf-Nonius B.V.
P.O. Box 12080
Vareseweg 127
3004 GB Rotterdam
The Netherlands
Tel: +31 (0)10 – 20 30 600
Fax: +31 (0)10 – 20 30 699
info@enraf-nonius.nl
www.enraf-nonius.com

Part number: 1498756_42
October 31, 2013

4-Series

Information Booklet
Informations-Booklet
Brochure d'information
Opuscolo informativo
Folletto de información
Informatieboekje

CONTENTS

1	Introduction.....	4
2	Einleitung	6
3	Introduction.....	8
4	Introduzione	10
5	Introducción.....	12
6	Introductie	14

1 Introduction

The 4-Series is a product line for physical therapy, comprising the following devices:

Model Number	Device	Description
0498900	Sonopuls 490	Ultrasound device
0498910	Sonopuls 492	Combined Ultrasound and Electrotherapy device
0498920	Endomed 482	Electrotherapy device
0498950	Vacotron 460	Vacuum device

The ultrasound devices can be supplied with one or two ultrasound applicators, depending on the article ordered. See the table in the section “package contents” for details.

Do not install and use this device before consulting the user manual, which is available on the CD-ROM included in this package (1). The Vacotron 460 has been designed for use with the Sonopuls 492 or Endomed 482. For details concerning its installation and operation, consult the user manual of these devices.

(1) On request the user manual can be obtained as a printed version.

CAUTION:

- Federal law (USA only) restricts this device to sale by, or on the order of, a physician or licensed practitioner. This device should be used only under the continued supervision of a physician or licensed practitioner.
- This device has been designed to be used only by or under the supervision of persons using the medical device in the course of their work and in the framework of a professional healthcare activity, who understands the benefits and limitations of electrotherapy and ultrasound therapy. I.e. “professional users”.
- For safe use of this device first familiarize yourself with the contents of the user manual, which is available on the CD-ROM included in this package.
- Read, understand and practice the precautionary and operating instructions. Know the limitations and hazards associated with using any electrical stimulation device. Observe the precautionary and operational decals placed on the unit.
- Before administering any treatment to a patient you should familiarize yourself with the operating procedures for each mode of treatment available, as well as the indications, contraindications, warnings and precautions. Consult other resources for additional information regarding the application of electrotherapy and ultrasound therapy.
- In case of ingress of liquids, unplug the unit from the mains supply and have it checked by an authorized person (see the paragraph on maintenance in the user manual).
- This device should be kept out of the reach of children.

Package Contents:

The package contents depends on the device model, which can be found on the packaging label. The following models are known:

Device Model:

Device Model	Sonopuls 490 (0498900)	Sonopuls 492 (0498910)	Endomed 482 (0498920)	Vacotron 460 (0498950)	Ultrasound treatment head large (1630905)	Ultrasound treatment head small (1630915)
1498901	•				•	
1498902	•					•
1498903	•				•	•
1498911		•			•	
1498912		•				•
1498913		•			•	•
1498920			•			
1498950				•		

The devices are packed with the following accessories:

Ultrasound models can be supplied with one or two ultrasound applicators:

1630905 Ultrasound Head Large - NCS S7010-R90B

1630915 Ultrasound Head Small - NCS S7010-R90B

Standard accessories 4-Series:

1498010 Device base (inclination support) (not for Vacotron 460)

3440001 Screwdriver

3444290 Power cord 250V/10A Europe-3x1 mm² 2.5 meter black

1498756 4-Series Information Booklet

1498757 4-Series Operating Instructions (cd rom)

Standard accessories ultrasound:

0167154 Information sheet ultrasound gel

0167314 Information sheet Mounting US Head Holder(s)

1498011 Holder for Ultrasound Head 4-Series - 1 for each ultrasound applicator

3442929⁽¹⁾ Contact-gel, bottle 250 ml, 1x

⁽¹⁾ = The Sonopuls is delivered with 1 bottle of contact-gel. The article-number 3442929 however represents a box of 12 bottles.

Standard accessories electrotherapy:

1460266 Moist pads for rubber electrodes 6x8 cm, set of 4 pcs.

3444020 Strap 100x3 cm

3444021 Strap 250x3 cm

2 x 3444129 Rubber electrodes 6x8 cm, 2 mm female, set of 2 pcs.

2 x 3444211 Patient cable 2-core & 2 mm male plugs - black, with coloured clips

Standard accessories vacuum:

2 x 3444503 Vacuum electrodes Ø 60 mm, set of 2 pcs.

3444505 Sponges Ø 65 mm, set of 4 (for vacuum electrodes Ø 60 mm)

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